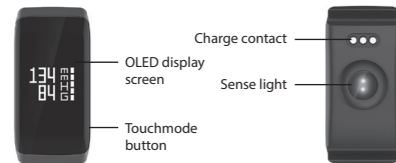
## Introduction

Activeband PRO MT854 is designed to be worn on dominant wrist and connects to Android or iOS device to help you monitor significant health parameters like Blood Pressure (BP), Heart Rate (HR), Blood Oxygen (O2). It also keep track of your daily activities like quantity of steps you've taken, length of sleep, overall fatigue and much more.

Device has a vibrate function which notifies you of alarms as well as incoming messages and calls. Device can be used to control camera on Android device. Vibrate alarm will prevent to lost your smartphone and also there is function which help to find connected smartphone by releasing ringtone.

ActivebandPro requires Android system 4.4 or above or iOS 8.0 or above. Make sure that your mobile device supports the Bluetooth 4.0 low energy standard (BLE).

## Device overview

### Note:

Before first use please charge device approximately for at least 30 minutes. If device is not used for longer time please charge it once a week.

### Charger connecting

Connect charging clips to charging contacts (the charging pins must touch contacts). The USB A type plug connect to normal USB port or charger with DC5V 1A.

### Note:

Fast chargers is not supported.

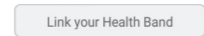
### Using device

Wrap device around your wrist medium tightly, about 3 cm from the wrist, to stick the sensor to skin surface.

Turn on the device by touching mode button under the screen. Using the same button you can switch modes.

To make initial setting (time, date, etc.) you have to install the "Wearfit" app form Android Google Play store or App Store for iOS.

Turn on Bluetooth interface in smartphone. Click "Link your Health Band" and choose correct device name from the list.

### Available modes:

-  1. Time mode: display time, battery and date
-  2. Step mode: automatically records number of steps
-  3. Distance mode: shows calculated walked distance
-  4. Calories mode: shows calculated burned calories
-  5. Heart rate monitor: automatically measure current heart rate (start displaying measurement after 10 seconds and shows averaged result after 60 seconds confirmed with vibration)
-  6. Blood oxygen monitor: automatically measure current quantity of oxygen and shows in %
-  7. Blood pressure monitor: automatically measure diastolic and systolic blood pressure. Measurement process takes up to 150 seconds and finishes with vibration.

**Note. Distance between band and skin, dirt or sweat on sensor, heavy hair can influence the measurement quality. Please be noted that blood pressure monitor in the device will not replace the medical equipment and cannot be equal with professional measuring device.**

-  8. Sleep monitoring mode: automatically recognizes activity status and monitor whole sleep progress by analyzing deep and light sleep hours
-  9. Stopwatch mode: touch and hold for 5 seconds mode button to switch to stopwatch mode. Touching will start and stop time counting. Holding the mode
-  10. Smartphone finder mode: touch and hold for 5 seconds mode button. Touching again will trigger the smartphone ringing call and vibration what help to find the device. Touching again will stop triggering. Holding the mode button will return to modes selection.

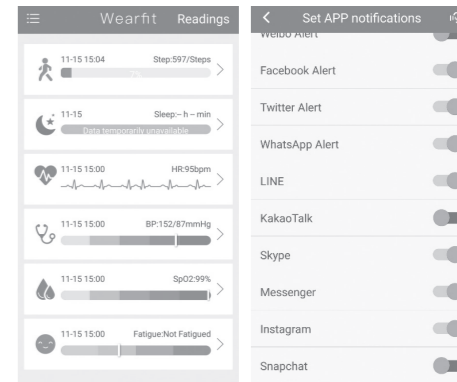
### Wearfit APP

When device is linked to Wearfit APP provides a lot of data about your daily, weekly and monthly activities. Also shows history of monitored health parameters.

Option "Wrist activation" can be enabled to shows time screen everytime you lift the wrist.

Device in connection with APP provides also:  
Alarm clock (intelligent reminders), incoming call or message notification

shows calling/sender number or name and releasing vibration. Notification of social media or communicators incoming entry.

*Due to continuous development specification and appearance of product are subject to change without prior notice. For technical support please visit [www.media-tech.eu](http://www.media-tech.eu).*

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